**DISINFECTANTS** 

# **AMITY HP75**


### PEROXIDE BASED SPORICIDAL HIGH LEVEL DISINFECTANT

Amity HP 75 is a powerful peroxide based high level disinfectant Ready-to-Use solution for use in Automated Endoscope Reprocessors (AERs), other automated decontamination systems, and to be used in aerial decontamination equipment.

It is suitable for the disinfection of flexible endoscopes, medical devices. Amity HP 75 Solution is registered as Class IIB Medical Device

## Primary features attributable to Amity HP75 are:

- · Primary features attributable to Amity HP 75 are;
- Active ingredient: Hydrogen Peroxide (HP);
- Glutaraldehyde free;
- No noxious odours or irritating effects;
- Reuse Period: Single use;
- HLD claim: 10 to 20 minutes at 30°C;
- No Activator Required;
- Corrosion Inhibitors: Yes.
- Complies with NFT 72-281 (2009)

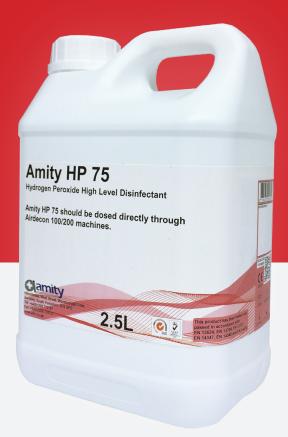

#### **Packaging**

- 2.5 litre containers



Guaranteed protection and performance, giving total infection control - safely

#### **Application**

Amity HP 75 is intended for HLD of critical and semi-critical, heat sensitive medical devices in AERs.

Amity HP 75 is intended for single use at  $30^{\circ}\text{C}$  or until the HP is below the MEC of 850ppm.

Amity HP 75 is intended for use in aerial decontamination equipment (Airdecon 200) for all Whole Room Decontamination Applications.

Please read and follow the instructions for use and precautionary statements contained on the label and MSDS. Amity HP 75 HLD is suitable for use on all types of flexible endoscopes, heat sensitive instruments and medical devices.

List of compatible materials:

Metals	Plastics	Elastomers
302 Stainless Steel	Teflon	ABS
316 Stainless Steel	Polyester	Viton A
410 Stainless Steel	Polystyrene	Nitrile
Chromium Plate (on steel)	Polycarbonate	Silicone
Inconnel	Polypropylene	Neoprene
Aluminium	Polyethylene	Natural Rubber
	Acrylic	
	PVC	
	Nylon 6, 6-6, 12	

The compatibility of Amity HP 75 has been carried out by conducting intensive exposure tests with the use solution. In addition biocompatibility testing has been conducted in order to demonstrate the rinsing of the chemical components from the flexible endoscopes.

#### **Amity HP75 Properties**

- Active ingredient: Hydrogen Peroxide (HP);
- Glutaraldehyde free;
- No noxious odours or irritating effects;
- Reuse Period: Single use;
- HLD claim: 5 minutes at 20°C;
- No Activator Required;
- Corrosion Inhibitors: Yes.

Sporicidal, Mycobactericidal, Virucidal, Bactericidal and Fungicidal. Tested to EN14885 for disinfection properties, EN 1040, EN 13624, EN 13704, EN 13727, EN 14347, EN 14348, EN 14476 & NFT 72-281 (2009).

Active ingredients: Hydrogen peroxide.





Amity Limited, Libra House, West Street, Worsbrough Dale, Barnsley, South Yorkshire, S70 5PG, UK

**T:** +44 (0) 1226 770787 **F:** +44 (0) 1226 770757 **E:** sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2018

DISTRIBUTED BY: