

# Single Use Spill Kit

## Emergency High Level Disinfection & Cleaning for ALL types of Body Fluid Spillage

**Guaranteed protection and performance, giving total  
infection control - safely**

**Primary features attributable to Single Use Spill Kit are:**

- High Level Disinfection and cleaning.
- Does not contain chlorine or peracetic acid.
- Complete package.
- Effective against bacterial spores e.g. Clostridium difficile (C dif), Bacillus cereus etc, and all blood borne viruses.
- Effective over a wide range of pathogens including mycobacteria, viruses, bacteria, fungi, yeasts and moulds, including MRSA, Hepatitis B, C and HIV.
- Effective against the difficult non-enveloped type viruses such as Polio, Adenovirus and Norovirus.
- Non-fixative.
- Non-corrosive - will not damage delicate coatings.
- No activation required.
- Cost effective.
- No fumes.
- Tested to EN14885 for disinfection properties



# Single Use Spill Kit

Emergency High Level Disinfection & Cleaning for ALL types of Body Fluid Spillage

## General

For ambulances, hospital wards, departments, laboratories, clinics, doctors surgeries, pharmacies and nursing homes a sudden and unexpected spillage of blood or any other body fluid, is both distressing and potentially dangerous.

By using a Spills Kit the hazard can be dealt with quickly, efficiently and above all safely. Our Biohazard Spill Kits are designed to enable staff in many walks of life to conform to the recommendations of the Department of Health for the safe collection and disposal of blood and blood-stained body fluid spills.

## Packaging

### Ready to Use

- 50ml Virusolve+ High Level Disinfectant / Cleaner
- 20g Virugel absorbent granules
- Gloves
- Apron
- Scapers
- Biohazard disposal bag
- Hand wipe
- Instructions for use



## Spill Kit Properties

| Active Against | Standards                             | Contact Time          |
|----------------|---------------------------------------|-----------------------|
| Spores         | EN13704<br>EN14347                    | 2 Minutes<br>1 Minute |
| Mycobacteria   | EN14348                               | 1 Minute              |
|                | Mycobacterium terrae                  | 1 Minute              |
|                | Mycobacterium tuberculosis            | 1 Minute              |
|                | EN14563                               | 5 Minutes             |
| Bacteria       | Mycobacterium terrae                  | 5 Minutes             |
|                | Mycobacterium avium                   | 5 Minutes             |
| Viruses        | EN1276, EN13727,<br>EN13629, EN14561  | 1 Minute<br>5 Minutes |
|                | EN 14476                              | 5 Minutes             |
|                | Polio, Adeno & Noro<br>HIV, Hep B & C | 1 Minute              |
| Fungi / Yeasts | EN1275, EN1650                        | 1 Minute              |
|                | EN13624, EN14562                      | 5 Minutes             |

## Application

This single use kit can be used on any type of body fluid spillage without the risk of fumes or damage to equipment.

### Bio-hazard spill kit instructions;

- 1 Put on gloves, mask and apron
- 2 Spray Virusolve+ onto spillage and leave for 2 minutes.
- 3 Pour sachet of Virugel onto affected area, ensure complete coverage. Leave for 2 minutes
- 4 Scrape debris into yellow bag using the scoops provided.
- 5 Spray surface with remaining Virusolve+ and wait 2 minutes.
- 6 Wipe surface clean and dispose of all products & packaging into yellow bag
- 7 Wipe hands using the alcohol free hand wipe, place this in Yellow bag and dispose of as per your health & safety guidelines.

**amity**  
international

· HEALTHCARE ·

Amity International,  
Libra House, West Street,  
Worsborough Dale,  
Barnsley, S70 5PG, UK.  
UK Tel: 0845 3707700  
Int Tel: +44 (0) 1226 770787  
Email: [sales@amityinternational.com](mailto:sales@amityinternational.com)  
[www.amityinternational.com](http://www.amityinternational.com)

Distributed by:

