

DISINFECTANTS

VIRUSOLVE® + EDS RTU

READY TO USE HIGH LEVEL DISINFECTANT / COLD STERILANT FOR ENDOSCOPES & MEDICAL DEVICES

Virusolve® + EDS RTU is a superior high level disinfectant (HLD) (Sterilant) that is designed for the decontamination of endoscopic, surgical and dental instruments & medical devices.

Virusolve® + EDS RTU is a patented proven HLD / Sterilant. The formulation does not contain any hazardous aldehydes or chlorine generating components. It is highly effective as a HLD / Sterilant against bacterial spores, mycobacteria, bacteria, fungi and viruses. The product is designed to eliminate the risk of crossinfection from spores, mycobacteria, viruses, fungi and bacteria.

Virusolve® + EDS RTU is safe to use on ALL flexible and rigid endoscopes.



Primary features attributable to Virusolve® + EDS are:

- High Level Disinfectant / Cold Sterilant
- Ready To Use
- No instrument damage
- Excellent material compatibility
- Safe
- Odourless
- Non-fixative
- Non-corrosive
- 5 minute soak at 20 °C
- No activation required
- Cost effective
- Tested to EN14885 for disinfectant properties

Packaging

- 5 L container



Virusolve® + EDS RTU is a superior, patented product for the decontamination of medical, surgical, dental and endoscopic instruments.

DISINFECTANTS

Application

Reprocessing Of Instruments - High Level Disinfection

For ALL critical and semi-critical instruments / medical devices including heat sensitive instruments:

For Manual Disinfection of instruments and equipment:


Stage 1:

• Make sure that instrument has been thoroughly cleaned (with Viruzyme family or Virudet N) and rinsed.

Stage 2:




1. Fill soaking tray with Virusolve®+ EDS Ready to use Solution

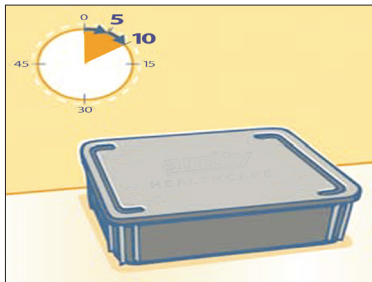

2. Soak for 5 minutes making sure that all internal channels are flushed through. This ensures bactericidal, fungicidal, virucidal, mycobactericidal and sporicidal performance is attained.

Note: Ensure that all spaces and recesses are completely filled with Virusolve®+ EDS fluid and no air pockets exist

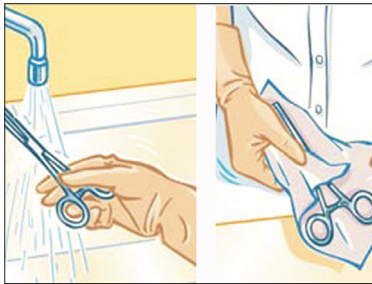

3. Rinse in sterile water and dry with sterile material

The medical device is now ready for use

Re-useage for Disinfection

Virusolve®+ EDS solution has demonstrated efficacy in the presence of organic soil contamination and microbiological burden during reuse. The active constituents' concentration of Virusolve®+ EDS solution during its use-life must be verified by the Virusolve®+ EDS Concentration Test Kit or Test Strips. Virusolve®+ EDS must be discarded after 14 days even if the active constituents are at or above the MEC. For MEC please refer to PIBS 08-006.

VIRUSOLVE®+ EDS RTU

READY TO USE HIGH LEVEL DISINFECTANT / COLD STERILANT FOR ENDOSCOPES & MEDICAL DEVICES

Virusolve®+ EDS Microbiological Properties

Active Against	Standards	Contact Time
Spores	EN13704 EN14347	2 Minutes 1 Minute
Mycobacteria	EN14348 Mycobacterium terrae Mycobacterium tuberculosis EN14563 Mycobacterium terrae Mycobacterium avium	1 Minute 1 Minute 5 Minutes 5 Minutes
Bacteria	EN1276, EN13727, EN14561	1 Minute 5 Minutes
Viruses	EN 14476 Polio & Adeno HIV, Hep B & C	5 Minutes 1 Minute
Fungi / Yeasts	EN1275, EN1650 EN14562, EN13624	1 Minute 5 Minutes

Active ingredients: Bis (3-aminopropyl) dodecylamine.
Didecylidimethyl ammonium Chloride



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 770787

F: +44 (0) 1226 770757

E: sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2018

DISTRIBUTED BY: