

DIBI HYDRA SYSTEM

Product description	Code	Content
Ultra-Comfort Quenching Cleanser	HS0033	300 ml bottle
Quenching Tonic Lotion	HS0133	300 ml bottle
Intense Rehydration Serum	HS0014	30 ml bottle
Deep Rehydration Cream Long-Lasting Effect	HS0118	50 ml jar
Long-Lasting Rehydration Fluid	HS0018	50 ml bottle

ACTIVE INGREDIENTS

HydraFil™, Imperata Cylindrica, Hyaluronic Acid, Trace Elements, Shea Butter, Avocado Oil.

DIBI EYE CONTOUR

Product description	Code	Content
Fluid	N1015J	15 ml bottle
Mask	N1030	30 ml tube
Cream	N0015J	15 ml tube

ACTIVE INGREDIENTS

Panthenol, NMF, Distilled Linden and Cornflower Water, Plant DNA, Plant Tensor, Collagen from Biotechnology, Borage and Camellia Oil, Gamma Oryzanol, Orange Flowers, Hyaluronic Acid, Pineapple and Papaya Extract.

DIBI Hydra System

DEEP REHYDRATION TREATMENT with DibiSkin Ultra and DibiSkin Power equipment

EQUIPMENT

The integration and synergy between DibiSkin Ultra and DibiSkin Power, even in the rehydrating program, enables a truly complete and effective beauty treatment to be performed, essential to ensure excellent results that are immediately visible and last over time.

ACTIONS

The innovative **DibiSkin Ultra** technology: 3 unique actions in only 1 treatment using a completely new system of ultrasound and ionophoresis. The two different probes enable even the most delicate areas, such as the areas around the lips and eyes, to be treated.

- **Cleansing:** in synergy with the **Quenching tonic lotion**, it eliminates devitalized cells and leaves the skin more receptive.
- **Permeation:** calibrated micronization of the active ingredients in the **Intensive rehydrating serum** to increase hydration deep down and on the surface. Their absorption is assisted by the ultrasound "stimulation" and boosted by the ionophoresis.
- **Patting:** stimulation of the microcirculation, working in synergy with the **Intensive rehydrating serum**, by means of light patting movements that stimulate the blood supply. A massage that also works on the muscle fibres, makes the facial tissues more turgid and creates an immediate lifting effect.

The treatment can also be performed with Dibi Four Seasons.

Using the advanced technology of electroporation, **DibiSkin Power** causes new temporary channels of access to be formed in the epidermis, through which the moisturizing active ingredients in the **Intensive rehydrating serum** can pass to boost the rehydrating effects to a maximum, both deep down and on the surface. For an immediately visible, superb filling effect. Moreover it creates a dermic reservoir-effect, storing the active ingredients in the cutaneous tissues, thereby prolonging and enhancing the results of the treatment.

The treatment can also be performed with Dibi Liftech.

EFFECTS

FULLY MOISTURIZED SKIN. Naturally **REHYDRATED DEEP DOWN** and **ON THE SURFACE:** the skin is fresher, smoother and extraordinarily soft to the touch. With a sensation of absolute comfort.

INDICATED FOR

All skins that need **INTENSE HYDRATION.**

FREQUENCY

Only after a careful check-up it is possible to determine the frequency of the professional treatments.

DURATION OF THE TREATMENT

About 1 hour.

MOD1145I



Distributed by: GTS GROUP S.p.A. • Viale Lombardia, 67 • 24020 Torre Boldone (Bg), Italy
tel. +39.035.4283350 • fax +39.035.362345 • www.gtsgroup.it • export@gtsgroup.it

American branch: DIBI INC. • 60 East 42nd Street, Suite 1502 • NY 10165, New York, USA
tel. +1.212.4909777 • 800.959.DIBI • fax +1.212.2860140 • www.dibi-usa.com • info@dibi-usa.com



The new dimension of beauty



Carefully remove all traces of make-up with **DECODERM CLEANSING LOTION** and **EYE&LIP MAKE-UP REMOVER**.

DIBI HYDRA SYSTEM

Product description	Code	Content
Ultra-Comfort Quenching Cleanser	HS0033	300 ml bottle
Quenching Tonic Lotion	HS0133	300 ml bottle
Intensive Rehydrating Serum	HS0218	50 ml bottle
Intensive Rehydrating Mask	HS0130	150 ml tube
Deep Rehydration Cream Long-Lasting Effect	HS0230	150 ml tube
Long-Lasting Rehydration Fluid	HS0018	50 ml bottle

ACTIVE INGREDIENTS

HydraFil™, Imperata Cylindrica, Hyaluronic Acid, Trace Elements, Shea Butter and Avocado Oil.

DIBI EYE CONTOUR

Product description	Code	Content
Fluid	N1015J	15 ml bottle
Mask	NP2050J	50 ml tube
Cream	NP1050	50 ml tube

ACTIVE INGREDIENTS

Panthenol, NMF, Distilled Linden and Cornflower Water, Plant DNA, Plant Tensor, Collagen from Biotechnology, Borage and Camellia Oil, Gamma Oryzanol, Orange Flowers, Hyaluronic Acid, Pineapple and Papaya Extract.



1



Carefully remove all traces of make-up with **DECODERM CLEANSING LOTION** and **EYE&LIP MAKE-UP REMOVER**.

2



Massage the **ULTRA-COMFORT QUENCHING CLEANSER** onto the face, neck and décolleté, and cleanse using disposable gloves moistened with warm water.

3



Pour the **QUENCHING TONIC LOTION** into a bowl and apply the product starting from the décolleté. Start up the **CLEANSING PHASE** and use the probe over the entire area to be treated following the **DIBISKIN ULTRA** method for 10 minutes. Use the small probe for the nose, chin, lip and eyebrow areas.

4



Apply approximately 4 ml of **INTENSIVE REHYDRATING SERUM** over the entire area to be treated starting from the décolleté; start up the **PERMEATION PHASE** following the **DIBISKIN ULTRA** method for 10 minutes. Apply the **NUOVAPELLE EYE CONTOUR FLUID**, use the small probe for the area around the eyes, lips and wrinkles.

5



Apply approximately 4 ml of serum over the entire area to be treated starting from the décolleté, and start up the **PATting PHASE** following the **DIBISKIN ULTRA** method for 10 minutes. Use the small probe for the area around the eyes and lips and the wrinkles. Then apply the **NUOVAPELLE EYE CONTOUR MASK**.

6



Apply a thin layer of **INTENSIVE REHYDRATING MASK** onto the face, neck and décolleté with a brush and leave on for 15 minutes.

7



Remove the **INTENSIVE REHYDRATING MASK** with the disposable gloves. Apply the **NUOVAPELLE EYE CONTOUR CREAM**.

8



Finish the treatment by applying and massaging in the **DEEP REHYDRATION CREAM LONG-LASTING EFFECT** or the **LONG-LASTING REHYDRATION FLUID** (selected according to the requirements of the skin).

9



Recommend the most suitable **HYDRA SYSTEM** products for home treatment.

1



Carefully remove all traces of make-up with **DECODERM CLEANSING LOTION** and **EYE&LIP MAKE-UP REMOVER**.

2



Massage the **ULTRA-COMFORT QUENCHING CLEANSER** onto the face, neck and décolleté and cleanse using disposable gloves moistened with warm water.

3



Dab the **QUENCHING TONIC LOTION** onto the face, neck and décolleté with two cotton balls.

4



Apply approximately 8 ml of **INTENSIVE REHYDRATING SERUM** with a brush over the entire area to be treated, aiding absorption by making circular movements with the **DIBISKIN POWER** equipment for a maximum of 10 minutes.

5



Apply a few drops of **NUOVAPELLE EYE CONTOUR FLUID** around the eyes; then apply the **NUOVAPELLE EYE CONTOUR MASK**. Apply a thin layer of **INTENSIVE REHYDRATING IMASK** with a brush onto the face, neck and décolleté and leave on for 15 minutes.

6



Remove the **INTENSIVE REHYDRATING MASK** with the disposable gloves. Apply the **NUOVAPELLE EYE CONTOUR CREAM**.

7



Complete the treatment by applying and massaging in the **DEEP REHYDRATION CREAM LONG-LASTING EFFECT** or the **LONG-LASTING REHYDRATION FLUID** (selected according to the requirements of the skin).

8



Recommend the most suitable **HYDRA SYSTEM** products for home treatment.