



QUICK SET UP GUIDE FOR THE LIVE LINK MOVEMENT SENSOR WITH PAGER/TRANSMITTER OPTION


- 1.** Your Movement Sensor with Transmitter and Pager
(Average range 80m: Please test before use!)

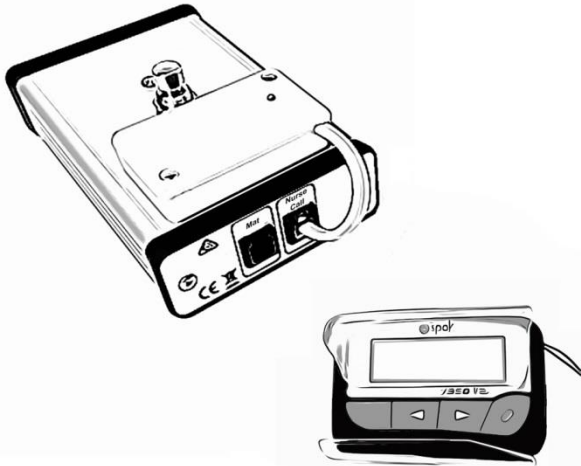


- 2.** Position sensor in the area to be monitored
(ensure there are no obstructions)

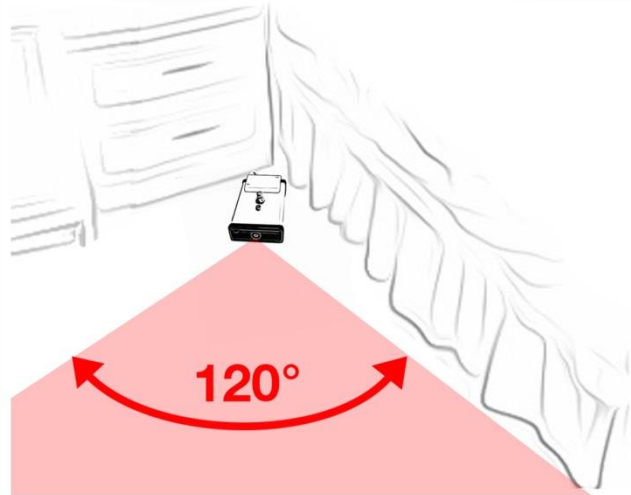

- 3.** To set the movement sensor push down on the button **once**

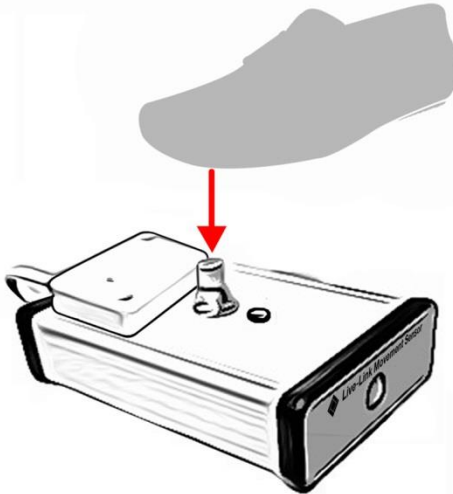

- 4.** The LED light will flash for **20 Seconds** to allow time for staff to exit the room

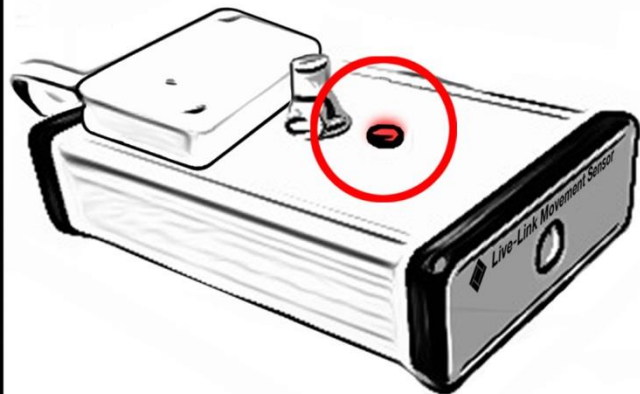

- 5.** When resident triggers sensor it will activate the pager



- 6.** To switch off sensor push down **once**.
See to resident & repeat process from step 3...



IMPORTANT WARNING!

The Movement Sensor is designed to be installed by the Care Staff.

If a user fails to position and switch on the sensor correctly, it will not pick up any movement.

It is the responsibility of the Care Staff to see that it is properly installed and tested with each use. Failure to test the Movement Sensor before each use, misuse, or inappropriate cleaning will void the warranty. These units are not a substitute for routine visual monitoring protocols and good care. There are clear guidelines on the set up of this product which must be followed. Failure to comply with these warnings may result in injury or death to the person in your care.

Live-Link Movement Sensor Troubleshooting guide

LIVE-LINK (CALL SYSTEMS) LTD.

ADMIN ADDRESS: 6 MILRIG HOLDINGS · KIRKNEWTON · EH27 8DE

Tel: 01506 884404 · Fax: 01506 884406

www.nursecallalarms.co.uk

livelink@nursecallalarms.co.uk

SYMPTOM	SOLUTION
The Red LED on the Movement Sensor is permanently on	<ol style="list-style-type: none">1. The 4xAA Batteries in the Movement Sensor need changing To change the batteries undo the two screws at the rear of the sensor and remove the cover, slide the metal plate underneath out and remove all four AA batteries, replace with new batteries.
The Pager is not being activated	<ol style="list-style-type: none">1. Is the Nursecall lead connected into the Nursecall socket on the sensor?2. Is the lead from the Sensor connected to your Nursecall system?3. If the Sensor is placed on cabinet/over bed tray – has the4. sensor been tested for correct height¹5. Is there anything blocking the sensor (i.e. crash mat, bedding etc)?6. Has the sensor been switched off after a recent activation?7. Has the Sensor been positioned correctly?
The Nursecall alarm goes off every 6 hours without activation	<ol style="list-style-type: none">1. The batteries need changing in the Sensor²
The Nursecall alarm goes off whilst the carer is still in the room	<ol style="list-style-type: none">1. The carer needs to leave before the red LED stops flashing – you have 20 seconds to leave the room after pressing the switch? Press the switch once to switch off and then press again to switch on ensuring you leave while the LED is flashing2. Is the Sensor connected to a chair/bed pad and the Resident has moved/got up? If so, check that the correct time delays have been set for that Resident¹.
The Nursecall alarm keeps being activated	<ol style="list-style-type: none">1. If connected to the chair/bed pressure pad/mat, check that the correct time delay settings have been made¹.2. Check that the Sensor has been placed in the correct position.3. The sensor beam has been activated by a carer while sensor is switched on. Sometimes this cannot be avoided due to where the sensor needs to be positioned i.e. across Residents bedroom door.

¹ see full set-up guide for steel chair/bed plate or vinyl pads

² see quick set-up guide