

LIQUICLEAN® N

Powerful Non-Enzymatic Cleaner for Removing Organic Soils from Surgical Instruments and Endoscopes

Liquiclean® N pre-soak cleaner has a powerful emulsifying action that assures complete cleanliness and free rinsing on all endoscopes and surgical instruments.

Liquiclean® N is extremely fast wetting, maintains high detergency and exceptional cleaning in hard, soft, cold or hot water.

Liquiclean® N is biodegradable and does not contain phosphates or NTA.

(pH 8.0 to 8.5 at use dilution).

- Removes organic soils from all surgical instruments, endoscopes
- For use as a manual pre-soak in hospitals and laboratories
- Biodegradable does not contain phosphates or NTA
- Hard water tolerant, effective in all water conditions
- No residue - no pyrogens

CAUTIONS:

Avoid prolonged contact with skin.
Do not swallow. Keep away from children.

Available in:

Cat No 345LQN 4 x 4 Litres



DIRECTIONS FOR USE

For Manual Soaking Only Endoscopes and Surgical Instruments

1. For light soil dilute at 4ml litre of water and heavy soil - dilute at 8ml litre of water.
2. Submerge instruments for 5 minutes for light soil and 10 minutes for heavy soil to remove all organic soils.
3. Rinse thoroughly with warm tap, distilled or sterile water.

Discard used solution

Liquiclean® N

Powerful Cleaner for Removing Organic Soils from Surgical Instruments and Endoscopes

Liquiclean® N pre-soak cleaner has a powerful emulsifying action that assures complete cleanliness and free rinsing on all endoscopes and surgical instruments.
LIQUICLEAN® N is extremely fast wetting, maintains high detergency and exceptional cleaning in hard, soft, cold or hot water.
LIQUICLEAN® N is biodegradable and does not contain phosphates or NTA (pH 8.0 to 8.5 at use dilution).

•Removes organic soil from all surgical instruments and endoscopes.
•For use as a manual pre-soak in hospitals and laboratories.
•Biodegradable does not contain phosphates or NTA.
•Hard water tolerant, effective in all water conditions.
•No residue - no pyrogens.

CONTENTS:
USA: 1 Gallon Export: 4 Liters
Made in U.S.A.

DIRECTIONS FOR USE:

For Manual Soaking Only
Endoscopes and Surgical Instruments
Manual Cleaning:
1. For light soil dilute LIQUICLEAN® N at a 4 ounce per one (1) gallon (4000L) of water.
For heavy soil dilute LIQUICLEAN® N at one (1) ounce per (1) gallon (4000L) of water.
2. Submerge instruments for 5 minutes for light soil and 10 minutes for heavy soil to remove all organic soils.
3. Rinse thoroughly with warm tap, distilled or sterile water.
* Use distilled water for tissue cultures.
Discard used solution.

CAUTION
DO NOT SWALLOW.
KEEP AWAY FROM CHILDREN.
AVOID SPLASHING OF CONCENTRATE IN EYES.
AVOID PROLONGED CONTACT WITH SKIN.
If accident occurs, flush eyes with water and call doctor.

08/2013
Chemclean Corporation
INDUSTRIAL CHEMICAL SPECIALISTS
130-45 180th Street, SPRINGFIELD GARDENS, NY 11454
TEL: (718) 338-2800 FAX: (718) 481-9470
1-800-“LET-CHEM” CLEAN WORK FOR YOU (1-800-538-2438)